


Examination of the syndesmosis

Shank in a.-p.-view


Device setup for the left leg

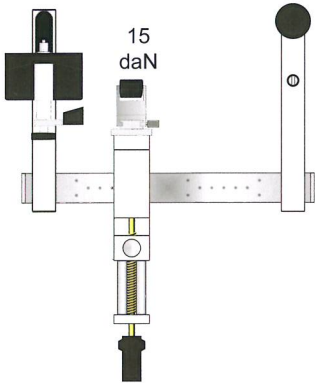


Please note

- Mount a counter support into the outer guide bush.
- First mount the anti-rotation lock and then the footholder on the opposite extension arm (see fig.).
- Instead of the front cushion install the adaptor with the pressure roll on the pressure device.
- The height of the pressure roll can be fixed with the fastening screw.
- Sit the patient with his knee approx. 20° flexed.
- Fix the heel with the swivel clamp by turning the fastening screw clockwise.
- Place the pressure roll approx. 5 cm above the lateral malleolus and anterior of the tibia.
- Pressure load for routine examination: 15 daN.

Device setup for the right leg

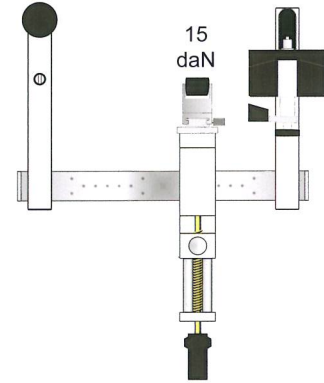

Positioning of the patient for the left leg

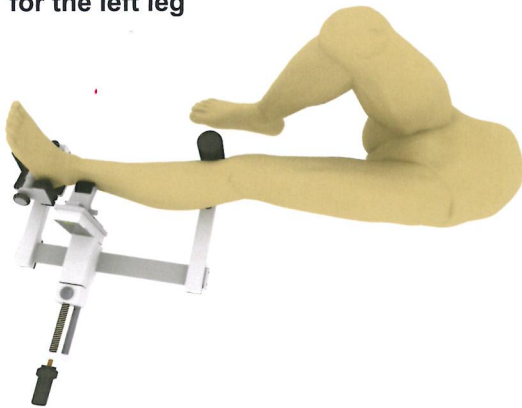

Positioning of the patient for the right leg

